

Tru-D iQ **Cleaning Instructions**

Tru-D SmartUVC, a PDI solution, recommends cleaning the Tru-D iQ System on a weekly basis or as needed according to your facilities protocols and standards.

Disconnect the iQ System from the wall or power source prior to cleaning. For additional system specific instructions on how to clean and maintain your system, see your IQ System Operations Manual and follow PDI product label instructions for use.

Controller

- Clean the Controller screen with PDI Easy Screen® Cleaning Wipes.
- Clean the outer shell and casters with PDI Super **Sani-Cloth**® germicidal disposable wipes.

Scout

- Clean the outer shell and casters with PDI Super **Sani-Cloth** germicidal disposable wipes.
- Clean the lamps with PDI Alcohol Prep Pads.

Tower

- Clean the outer shell and casters with PDI Super **Sani-Cloth** germicidal disposable wipes.
- Clean the lamps with PDI Alcohol Prep Pads.



Cleaning and disinfection were validated by PDI Super **Sani-Cloth** germicidal disposable wipes. Data on file. Compatibility study 2023.

Tru-D®

LEARN MORE AT TRU-D.COM



PDI Products Reorder Information

REORDER NO. WIPE SIZE CASE PACK CASE WGT CASE CUBE PALLET TI/HI



Super Sani-Cloth® Germicidal Disposable Wipe

Fast, 2-minute contact time allows for a quick room turnover. Ideal for daily use in fast-paced environments that require short contact times and broad coverage of microorganisms.

	REORDER NO.	WIPE SIZE	CASE PACK	CASE WGT	CASE CUBE	PALLET TI/HI
Large Canister	Q55172	6" x 6.75"	12/160s	25.63 lbs	1.430 ft	10/3
X-Large Canister	P86984	7.5" x 15"	6/75s	16.71 lbs	0.936 ft	10/4
Large Individual Packet	H04082	5" x 8"	10/50s	7.72 lbs	0.539 ft	20/4
X-Large Individual Packet	U87295	11.5" x 11.75"	3/50s	7.51 lbs	0.500 ft	24/3
Softpack	A22480	8.2" x 9.8"	9/80s	15.40 lbs	0.948 ft	10/4



Alcohol Prep Pads - Sterile

Provides rapid-acting skin antiseptics in a sterile formulation. 70% IPA.

	REORDER NO.	WIPE SIZE	CASE PACK	CASE WGT	CASE CUBE	PALLET TI/HI
Medium Prep Pad	B60307	1.1" x 2.6"	20/200s	12.40 lbs	0.84 ft	8/7
Large Prep Pad	C69900	1.7" x 3.5"	10/100s	7.20 lbs	0.48 ft	12/8



Easy Screen® Cleaning Wipe

A compatible touchscreen cleaner that works at the speed of technology.

	REORDER NO.	WIPE SIZE	CASE PACK	CASE WGT	CASE CUBE	PALLET TI/HI
Medium Canister	P03672	6" X 9"	12/70s	10.64 lbs	0.990 ft	8/5
Packet	H01050	5" X 8"	10/50's	5.84 lbs	0.483 ft	12/8

For more information, or to place an order, contact your local PDI sales representative.